



# WoPeeH<sup>®</sup>-pocket

Women's Peeing Help  
Urinary aid for women who stand up!



use it because you need to...  
because it is convenient...  
or just because you want it!

## Manual

### 1. Open WoPeeH (Fig. 1)

Remove the **cap** from the inlet opening;  
**pull** (1), **twist** (2) and **pull** (3) the front part until fully opened.

Figure 1

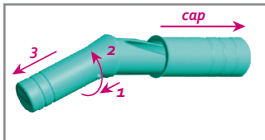


Figure 2

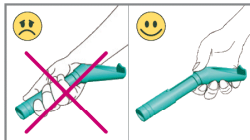


Figure 3



### 2. Open your pants

Open your zipper and push your panties aside or down  
(don't lower your trousers during use)

### 3. Place WoPeeH (Fig. 2)

Hold WoPeeH-pocket with one hand and slide it into your panties.  
Place the inlet opening securely fully against your body. Push the  
backside **just behind your vagina** against the skin.

### 4. Urinate (Fig. 3)

Direct the outlet down and straight forward and urinate. <sup>\*2)</sup>

### 5. Wipe dry

When you are done, slide the backside carefully along your body to  
remove left over droplets.

<sup>\*2)</sup> **If you are not able to urinate** it is possible you are pushing too hard  
or you may have placed your WoPeeH too far to the front:  
reduce pressure or slide your WoPeeH a little backwards and try again.

[www.wopeeh.nl](http://www.wopeeh.nl)

### Important!

**Note:** Please **open your WoPeeH gentle** because too much  
force may damage the pin in front part.

**Note:** We recommend that you try out WoPeeH-pocket in a  
familiar environment (say bathtub or shower).

**Note:** Pay attention to the **discharge of the last urine**.  
You may have to practice a few times before you are  
able to empty your bladder completely.

**Note:** WoPeeH-pocket is only suitable for **external use**.

**Note:** WoPeeH-pocket is **not** a children's toy!

### Cleaning your WoPeeH

It is recommended that you wash or rinse your WoPeeH  
and the cap immediately after use with clean water.  
If there is no clean water available: shake out well and close for  
transportation. Clean always as soon as possible. Use a mild  
detergent regularly.

### Storing your WoPeeH

If you don't use your WoPeeH for a longer period of time: clean  
your WoPeeH and the cap and dry well before storing. Store  
preferably in an opened position.

### Available colors



### WoPeeH-pocket

Is a urinary device that can be closed with a cap.  
Is meant for your personal hygiene  
Is easy to clean to ensure you can use it many times.  
Is made of an environmentally friendly plastic. <sup>\*2)</sup>

<sup>\*2)</sup> Polypropylene can be recycled and when burned causes  
no harmful emissions.

